

# **CardioMD IV**

## ***Cleaning***

Cleanliness is a prime rule in maintaining the equipment. At least once a month, wipe the external metallic surfaces to remove any foreign material that may have accumulated.

Enameled surfaces should be wiped using a clean cloth moistened slightly with a good quality mild cleaner.

## ***Disinfection***

It is important that any part of the equipment which comes into direct contact with patients is cleaned and disinfected on a regular basis. This applies particularly to the imaging table and mattress. Once a week, use a mild disinfecting agent such as alcohol or chlorhexidine (2% solution in water) to wipe all surfaces.